

PROVEN SHAPE CONICAL CONNECTION

BEGO Semados® SCX CC



"Made in Bremen"
quality
since 1890



Achieving success together



MAKE THE MOVE OF YOUR LIFE

BEGO Semados® SCX CC



“

Trust and safety are the basis of every treatment. In order to meet this requirement, we have specifically further developed our proven implant systems – with a new, deep-tapered connection that impresses with its outstanding tightness and stability.

The proven external geometry and our high-purity TiPure^{plus} surface are retained here. The result: a modern system for esthetically sophisticated restorations that is characterized by intuitive handling, high-quality components, and the highest manufacturing standards.

”

Susanne Grüttner, M.Sc., Oral Surgeon / Senior Clinical Specialist, BEGO Group

Dear practitioners,

Modern dentistry places extremely high demands on implant systems in terms of esthetics, functionality, and efficiency. The Semados® SCX remains a permanent fixture in the BEGO portfolio – and has now been further optimized. With the new Conical Connection portfolio, we present you with an innovative further development of our proven implant solutions that makes your daily work easier and offers you many advantages based on the latest findings.

At the heart of the system is the deep-tapered implant-abutment connection, which ensures a stable, gap-free interface. The reduction to just two internal geometries (M, L), coupled with intuitive prosthetic color coding, saves you valuable time and increases safety at your practice. The newly developed prosthesis screw also ensures uncomplicated and safe handling, as it cannot fall out when the abutments are removed thanks to an internal thread in the prosthetic abutments.

An outstanding feature of the BEGO Conical Connection implants is the extremely low vertical insertion depth, which ensures optimized stability and excellent accuracy of fit – even in challenging conditions. In addition, the risk of screws working loose has been significantly reduced, which speaks for the long-term safety and reliability of the implant restoration.

The prosthetics also remain easy to handle, despite the steep taper of the interface, and enable effortless, precise insertion of the prosthetic components without compromising the accuracy of fit. The innovative design of the internal connection prevents cold welding of the components, meaning that the prosthetic components can be easily detached again.

With the BEGO Security Plus warranty, we offer you additional security: Our 5-year warranty covers your fee, as well as the prosthetic restoration and material costs – offering a genuine advantage for you and your patients.

See for yourself over the following pages why the implants in the BEGO Semados® SCX range continue to enjoy a strong position among BEGO implants. Be inspired by the innovative benefits of our new Conical Connection portfolio!

Yours truly,

André Henkel

Team Lead Implants & Surgery
Category Management



As a practitioner, you are looking for reliable solutions that deliver optimal treatment results – Semados® SCX CC implants offer you stability, precision, and carefully considered handling.

👑 STABLE AND TIGHT CONNECTION

The deep-tapered internal connection ensures a gap-free fit and reduces tension in the implant thanks to the bionic thread.

👑 CAN REDUCE BACTERIAL COLONIZATION

The tight tapered connection can minimize the penetration of bacteria and thereby promote both tissue stability and implant health.

👑 PRESERVATION OF THE CRESTAL BONE AND OPTIMIZED ESTHETICS

A distinct platform switch supports the shaping of stable soft tissue for sophisticated esthetic results.

👑 EASY TO USE

The intuitive color coding and the two internal geometries (M, L) simplify prosthetic assignment.

👑 QUICK INSERTION

Our implants are designed so that they can be inserted with just a few turns. This saves you valuable time in the treatment room and ensures a precise fit.

👑 OPTIONAL NAVIGATED/TEMPLATE-GUIDED INSERTION

Using the BEGO Guide System, you can precisely and safely position Ø 3.25 – 4.5 implants, which leads to optimal outcomes and satisfied patients.

MANUFACTURED IN GERMANY TO SATISFY THE HIGHEST QUALITY REQUIREMENTS

Our implants are 100% developed and manufactured in Germany, giving you the confidence to offer your patients first-class quality and reliability.

ROYALLY IMPLANTED

One important way in which the BEGO Semados® SCX CC implants excel is through their suitability for use in a wide range of indications. Whether you are dealing with a single-tooth restoration, a bridge restoration, an immediate restoration, or positioning implants for a specific esthetic – SCX implants are designed so that you can deliver first-class results for your patients.

The implants are also exceptionally well suited for use in a large number of clinically challenging situations. Thanks to their diameter of 5.5 mm and length of 7 mm, the implants are also suitable for applications where minimal bone height is available. The success rate of our implants, even in challenging conditions, speaks for itself.

The BEGO Semados® SCX CC implants open up new opportunities for your practice and your patients. Discover the full potential of this innovative implant solution.

REDUCED COMPLEXITY

based on just two internal geometries

The intuitive color coding and the two internal geometries (M, L) simplify prosthetic assignment.

STABLE ABUTMENT FIT

thanks to 8° internal taper

The precise connection technology facilitates an accurate and stable fit of the prosthetic components in the implants.

FOR SUSTAINABLE AESTHETICS

through consistent platform switch

The pronounced platform switch shifts the interface away from the crestal bone and promotes a stable soft tissue cuff – the basis for long-term esthetic results.

SELF-TAPPING THREAD DESIGN

with optimum cutting angle

The innovative thread design enables easy insertion of the implants and excellent primary stability.

PARALLEL-WALLED IMPLANT SHAPE

with rounded apex

Simple to use – insertion with self-centering.

COVER SCREWS

included as standard and can also be specifically supplemented

Included in the scope of delivery or available separately in 2 mm height for special surgical techniques.



ROYAL TREATMENT

RETHINKING PROSTHETICS – FOR MAXIMUM EFFICIENCY AND PERFECT ESTHETICS

The new Conical prosthetic line combines clinical precision with intuitive application and esthetic perfection, even in challenging situations. With the new Conical prosthetics, we offer you a sophisticated system for all types of restoration. The components interlock seamlessly, support modern treatment procedures, and establish optimum conditions for implant restorations that are successful in the long term.

👑 CLEAR EMERGENCE PROFILE AND UP TO FIVE GINGIVAL HEIGHTS

Uniformly concave components and up to five gingival heights per connection facilitate targeted soft tissue guidance – for predictable esthetics, even in complex anatomical situations.

👑 INTUITIVE COLOR CODING FOR RELIABLE ASSIGNMENT

The clear color coding of all components (M = pink, L = gold) facilitates quick identification and reduces the risk of mix-ups in everyday practice and laboratory work.

👑 TWO INTERNAL CONNECTIONS - MAXIMUM OVERVIEW, LESS EFFORT

The fact that only two interfaces are used for all implant diameters ensures efficient stockkeeping, leaner ordering processes, and faster training for new team members.

👑 ANGLED SCREW CHANNELS – GREATER SCOPE, MORE CONTROL

The adhesive bases of the Conical system allow individual screw channel angulation – for screw-retained restorations, even with deviating implant alignment and located in the esthetically sensitive region.

👑 DIGITALLY READY – FOR MAXIMUM INDIVIDUALITY

The system is fully CAD/CAM-capable – with scannable components, digital analogs, and libraries for seamless processes from intraoral scan to final restoration.

👑 MULTI^{PLUS} FOR EVERY CASE – FROM FULL DENTURES TO SINGLE TEETH

The Multi^{PLUS} concept is not only suitable for bridges and bars, but also for single-tooth restorations – ideal for cases with limited bone availability or challenging implant positions.



ROYALLY

COMPREHENSIVE ORAL SOLUTIONS



INDIVIDUAL CAD / CAM-MANUFACTURED PROSTHETIC COMPONENTS

Our CAD/CAM-manufactured prosthetic components are precisely matched to the SCX CC implants and are produced at the largest owner-managed milling center in Europe – directly in Bremen. You will receive your customized restoration within 48 hours, reliably and precisely manufactured.

Whether scanned digitally or sent in conventionally: We offer you maximum flexibility, a wide range of materials, and tailor-made results for demanding clinical situations.

MULTI-UNIT CONCEPT

The BEGO MultiPlus system enables immediate restoration at post level, even with reduced bone volume and complex anatomy. The posts, which are supplied sterile in various angles and heights, create the basis for screw-retained bridge and bar restorations – including CAD/CAM-manufactured solutions from our milling center. Thanks to the screw connection, a partially removable prosthesis can also be realized – functional, durable, and suitable for further restoration at any time.




NAVIGATED SURGERY

The BEGO Guide System supports you in inserting implants with a diameter of 3.25 to 4.5 safely, precisely, and efficiently guided by a template. Whether tooth-supported or mucosa-supported with a scan prosthesis – the template adapts flexibly to your surgical procedure and enables minimally invasive, predictable work.

This means that you are ideally positioned, even without your own planning software: On request, the BEGO Guide Planning Center will take care of the complete planning and deliver the appropriate template directly to your practice. This allows you to increase process reliability, reduce operating times, and ensure reproducible results – with a system that integrates effortlessly into your workflow.

ROYAL

BEGO TRAYS

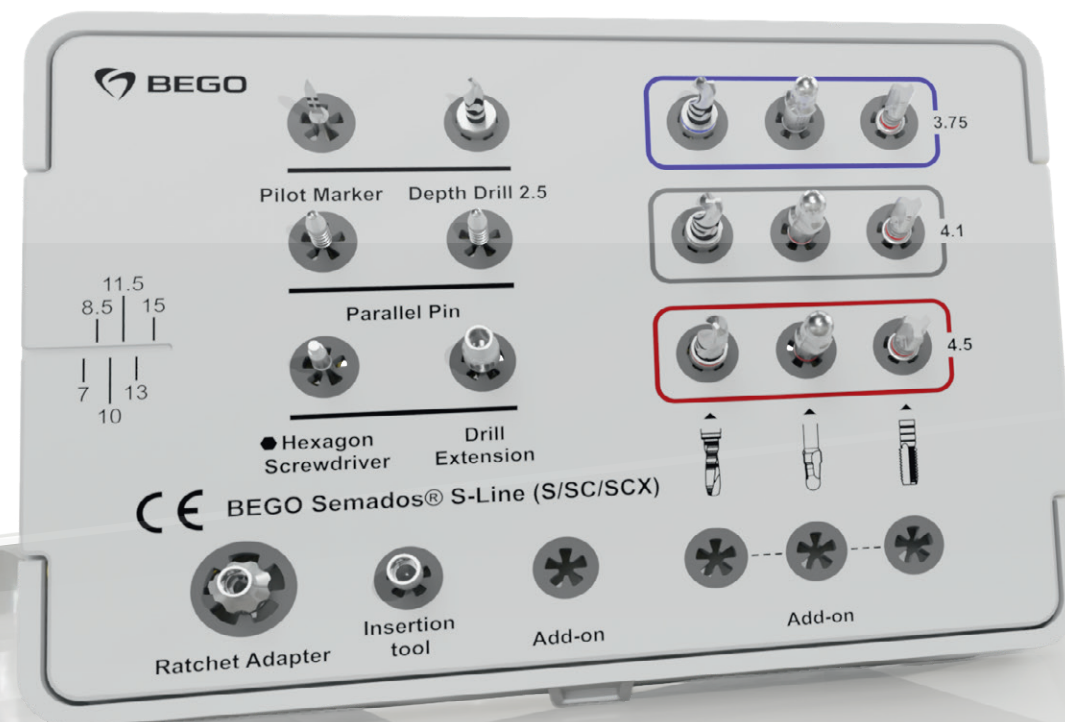

BEGO SEMADOS® COMPACT TRAY – EFFICIENCY WITHOUT COMPROMISE

Optimized for the main diameters used in implantology, the Compact Tray offers a reduced yet high-performance solution for the majority of conventional surgical procedures.

- 👑 Compact and focused – smaller than the Standard Tray and with all important instruments for Ø 3.75-4.5 ideal for targeted procedures
- 👑 Fast and precise – contains only the most important instruments for Ø 3.75 - 4.5 for targeted procedures
- 👑 Drill stops compatible – for maximum safety in depth control when using depth drills

THE BENEFIT FOR YOU

Less is more – get everything you need in a compact form.



BEGO SEMADOS® TRAY^{PLUS} - MAXIMUM FLEXIBILITY, SOPHISTICATED DESIGN

Designed and equipped for conventional surgery


- 👑 All-inclusive set containing all instruments (optional Ø 5.5 drill module)
- 👑 Perfect organization – clear arrangement for efficient processes
- 👑 Maximum precision – use of optional drill stops possible for exact depth control when using the depth drill

THE BENEFIT FOR YOU

Get everything you need – intelligently and clearly organized.

ROYAL

BEGO TRAYS

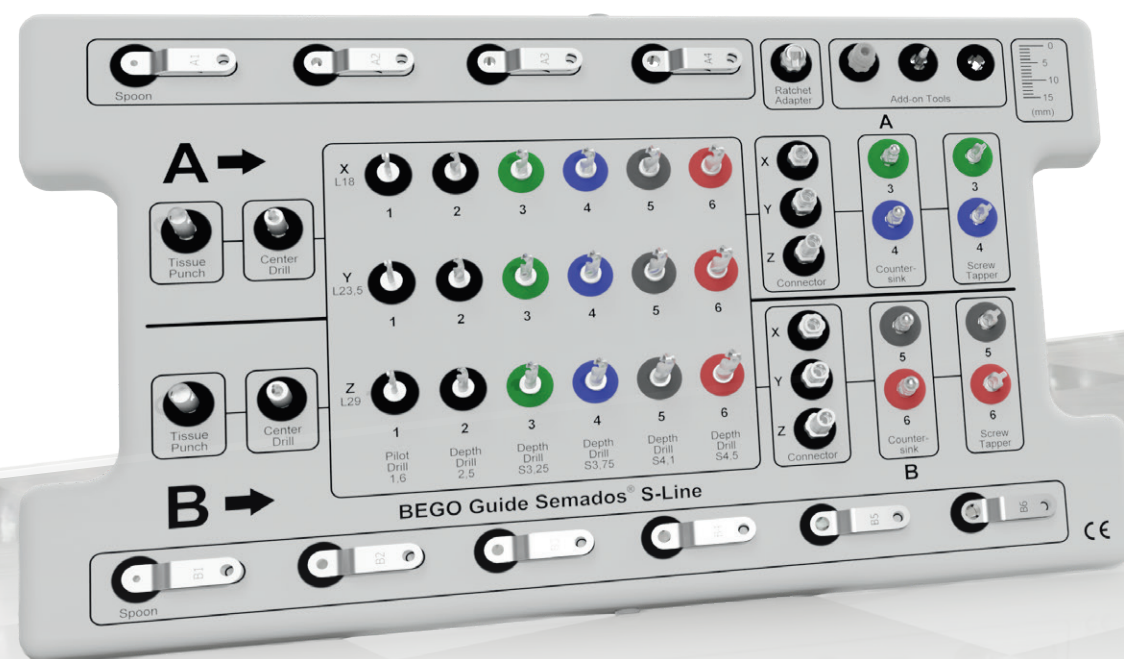


BEGO Guide Tray - Precision through navigation

The safe alternative for surgery: The BEGO Guide Tray ensures maximum control during template-guided preparation and implantation.

- 👑 Self-locking spoons – intuitive handling and stable hold
- 👑 Navigated preparation – for maximum precision and safety
- 👑 Fully equipped – all necessary instruments for Ø 3.25 - 4.5 included

THE BENEFIT FOR YOU

Top-level safety and control – for predictable, reproducible results.

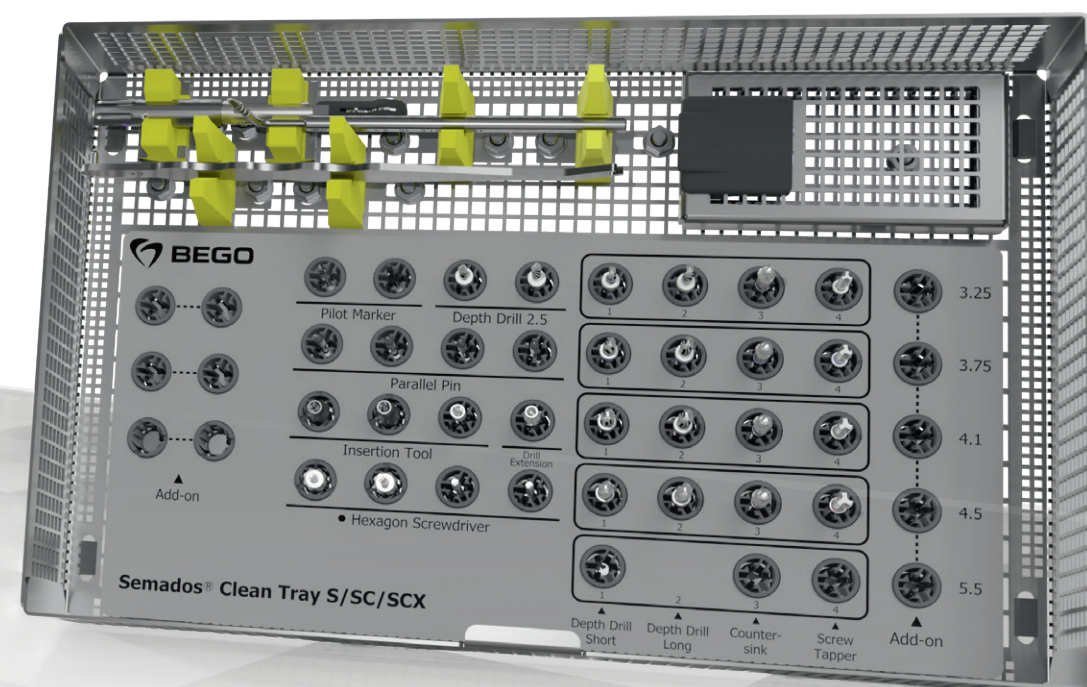

BEGO Semados® Clean Tray (Wash Tray) – Hygiene at the highest level

Specially developed for the requirements of mechanical preparation, the stainless steel BEGO Wash Tray is the ideal solution both for practices and large medical facilities.

- 👑 For holding and storing the matching BEGO Semados® Tray Set (all instruments and tools for equipping the Clean Tray, optional drill module Ø 5.5)
- 👑 Safe and efficient – reduced manual effort, less sorting, and minimized contact with contaminated instruments for smooth processes.
- 👑 Perfect cleaning – developed by the specialists at Aesculap for optimum cleaning

THE BENEFIT FOR YOU

Automated preparation without manual pre-sorting – for maximum hygiene, time savings, and safety in everyday clinical practice.


ROYAL

ORDERING


BEGO Semados® SCX CC implants

Description	Size	Length	Interface	REF
incl. cover screw and titanium insertion post, TiPure ^{Plus} surface, cold-formed titanium grade 4, sterile, 1 pc. 	3.25	L7	M	–
		L8.5		60323
		L10		60324
		L11.5		60325
		L13		60326
		L15		60805
incl. cover screw and titanium insertion post, TiPure ^{Plus} surface, cold-formed titanium grade 4, sterile, 1 pc. 	3.75	L7	M	–
		L8.5		60327
		L10		60328
		L11.5		60329
		L13		60307
		L15		60806
incl. cover screw and titanium insertion post, TiPure ^{Plus} surface, cold-formed titanium grade 4, sterile, 1 pc. 	4.1	L7	M	60331
		L8.5		60332
		L10		60333
		L11.5		60334
		L13		60335
		L15		60807



Accessories

Description	REF
Cover screw, Conical ICS TP, M - L2, 1 pc. 	60350


Spare parts

Description	REF
Cover screw, Conical ICS, M, 1 pc. 	60347


BEGO Semados® SCX CC implants

Description	Size	Length	Interface	REF
incl. cover screw and titanium insertion post, TiPure ^{Plus} surface, cold-formed titanium grade 4, sterile, 1 pc. 	4.5	L7	L	60336
		L8.5		60337
		L10		60338
		L11.5		60339
		L13		60340
		L15		60808
incl. cover screw and titanium insertion post, TiPure ^{Plus} surface, cold-formed titanium grade 4, sterile, 1 pc. 	5.5	L7	L	60341
		L8.5		60342
		L10		60343
		L11.5		60344
		L13		60345
		L15		–

Accessories

Description	REF
Cover screw, Conical ICS TP, L - L2, 1 pc. 	60351

Spare parts

Description	REF
Cover screw, Conical ICS, L, 1 pc. 	60348

BEGO Security Plus

Description	REF
BEGO Security Plus Implant warranty 	83538

ROYAL INFORMATION

With our
LiVE Service Center,
you receive fast and
personal service
also by
video call.

Dear customers and partners,

Our teeth have a significant effect on our well-being as humans – and replacing them comes with enormous responsibility. In the BEGO Group, this knowledge has been motivating us for over 135 years, and in implant dentistry for over 30 years, to ensure that patients feel safe and happy with our implant dentistry solutions.

OUR WAY OF HELPING YOU.

As a family-run enterprise, having a close relationship with our customers and maintaining strong personal collaboration are two of our most important goals. With us as your partner, you have access to a comprehensive product range supplemented by our unique BEGO Security Plus warranty and LiVE service program. Place your confidence in us, and you will not be disappointed – ever.

Good customer service is, first and foremost, characterized by its human component. At BEGO, more than 40 specialists from the Customer Service Center at the Headquarters in Bremen personally look after your interests. From product consultation, accepting orders, and advice on application to technical support, we are happy to be of help! Curious? Get in touch with us on: +49 421 2028-200.



You can find more info on our
service program at:
<https://www.bego.com/service/>



ROYAL WARRANTY

Place your trust in the BEGO brand and make the most of our comprehensive 5-year warranty. The BEGO Security Plus warranty covers not only your fees but also the prosthetic restoration and any material costs incurred*.



The warranty covers a maximum of EUR 2,500 per single implant restoration or up to a maximum of EUR 20,000 for a comprehensive restoration with multiple BEGO Semados® implants



No contractual obligation



Low administrative effort



Low costs



60 months (5 years) warranty from implant placement



Applies to restorations with BEGO Semados® implants, their BEGO prosthetic components, and prosthetic supraconstructions manufactured using the BEGO CAD/CAM process.



More info:
www.bego.com/security-plus





BEGO Implant Systems GmbH & Co. KG

Wilhelm-Herbst-Str. 1 · 28359 Bremen, Germany
Tel. +49 421 2028-246 · Fax +49 421 2028-265
E-mail order.imp@bego.com · www.bego.com

Want to keep up-to-date with all BEGO News?

Register here for the newsletter: www.bego.com/newsletter



CURIOUS?

Find more info at:
www.bego.com/de/queen-cc



Find more info on our
BEGO Security Plus warranty at:
www.bego.com/security-plus



Find more info on our
service program at:
www.bego.com/de/service/

Errors excepted and subject to changes. The provision of our deliveries and services are based on our general terms and conditions (T&C), which can be accessed at www.bego.com and forwarded to you upon request.

Please note that not all of our products and services are available in every country due to regulatory approval.

For further information, please contact your designated BEGO representative.

Images and illustrations in this flyer are examples only. The colors, symbols, design, and specifications on the labels and/or packages shown may deviate from real products.